amoena

RECOVERY CARE

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.

Purpose

Surface pressure is employed to prevent the formation of oedema and to support venous return and lymphatic drainage by increasing interstitial pressure. Compression therapy is usually a long-term treatment and requires appropriate products – as prescribed by a doctor if medically indicated – to be used by the patient themselves. In addition, Amoena Pressure Pads can increase the pressure in a targeted fashion via insertion into a garment, thus supporting oedema therapy in the case of specifically localised oedema. The breast shell supports the local mobilisation of fibrotic or oedematous tissue. Since the stage also determines the form of therapy, the Amoena Lymph Flow SB FC and Amoena Lymph Flow Long SB FC in Compression Class 1 (approx. 20 mmHG) are to be used for the following indications.

Indications

Damage to the venous/lymphatic vascular function/structure of the thorax and/or mamma (e.g. post-operative/post-traumatic, congestion due to immobilisation) with mild and moderate swelling, mild to moderate risk of oedema, feeling of heaviness, for oedema reduction, relief of discomfort and pain

Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV, decompensated heart failure, septic phlebitis, phlegmasia cerulea dolens, acute erysipelas, acute bacterial, viral or allergic inflammations, untreated or advanced skin diseases/contact eczematata, untreated open wounds, swellings or extremities of an unknown origin, untreated malignant tumours.

For compression textiles with a magnetic closure under the zip:

- Cardiac pacemaker
- Implanted cardioverter defibrillator (ICD)

Relative contraindications

Pronounced weeping dermatoses, malignant lymphoedema, peripheral arterial occlusive disease (PAOD) stage I/II, peripheral neuropathy in diabetes mellitus, sensory disorders of the extremities, compensated heart failure, skin infections, skin diseases (e.g. blistering dermatoses), intolerance to pressure or ingredients of the product, primary chronic polyarthritis. If there are open wounds in the treatment area, the wound area must first be covered with an appropriate dressing or bandage before the compression garment is applied. Amoena cannot accept any liability if these contraindications are not observed.

Side effects

No side effects are known if this product is used properly. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects. However, if negative changes such as skin irritation occur during prescribed use, please contact your doctor or specialist medical retailer immediately. If an intolerance to one or more components of this product is known, please consult your doctor before use. The manufacturer is not liable for any damage/injury caused due to improper handling or misuse.

Material composition

Textiles: Please refer to the care label for the exact percentage-based material composition. Products are free of latex and nickel.

Pressure pad: Polydimethylsiloxane, polyurethane film, polypropylene protective film (PP)

Breast shell: Polydimethylsiloxane, Talcum

General instructions for use

Your specialist fitter will be happy to show you how to apply the product correctly. Only wear your CuraLymph textiles, the pressure pad and breast shell as prescribed by your doctor. Rough, unfiled and cracked fingernails and/or sharp-edged rings etc. may damage the fine material. The compression garment, pressure pad and breast shell should only be dispensed by trained specialists at medical retailers. Any progression in therapy success or altered clinical picture will necessitate a new measurement and, if applicable, selection of the correct size. For this reason, it is advisable to have this checked by a specialist medical retailer. In the event of damage, please contact your specialist retailer in confidence. Please do not repair this product yourself, as this may impair its quality and medical efficacy. For hygienic reasons, it is not suitable for the care of more than one patient.

CURALYMPH TEXTILES

Lymph Flow SB FC, Lymph Flow Long SB FC

Application instructions for textiles

1. Place the compression textile flat on an underlay and open the front closure completely (Fig. 1).
2. Now slip one arm into the strap and then into the other strap. (Fig. 2)
3. Now pull the compression textile over your shoulders and fit it snugly. (Fig. 3)
4. Lymph Flow SB FC: Close the magnetic closure first and then pull the zipper from the bottom up to just above the cup area.
Lymph Flow Long SB FC: Close the hooks from the bottom up to just above the cup area. (Fig. 4)
5. To achieve correct positioning, reach into the cup with the opposite hand and lift your breast slightly (Fig. 5). Then close the compression textile completely. Adjust the straps to the desired length. (Fig. 6)
6. If needed, you can insert a prosthesis into the prosthesis pocket of the compression textile.

Care instructions for textiles

Please follow the care instructions on the sewn-in care label. If present, all fasteners must be closed. It is recommended that you turn the CuraLymph textiles inside out before washing. Wash the textiles separately the first time to avoid discolouration due to bleeding. Your compression garments should be washed daily on a gentle or delicate cycle at 40°C. A laundry net provides additional protection for your compression garments. Please do not use fabric softener! If washing by hand, please rinse generously and do not wring. The drying time can be shortened by placing the compression garment on a thick terry towel after washing, rolling it up tightly and squeezing it vigorously. Next, hang up your compression garment loosely. Please do not leave it in the towel and do not dry it on a heater or in the sun. The compression garment must not be dry-cleaned or tumble-dried.

LYMPH FLOW PRESSURE PAD

Application instructions for pressure pads

The pressure pad can be inserted under the garment textile in the required place. The textured surface is directed towards the skin. The pressure pad can be cut to according to your needs. It is supplied with a sample size of Amoena Soft Cleanser 30ml and an Amoena Soft Brush.

Care instructions for pressure pads

Please clean the pressure pad before each use, **especially before the first application.**

1. Apply three drops of Amoena Soft Cleanser to the dry, textured surface and then moisten the soft brush with lukewarm water. (Fig. 7).
2. Now clean the structured surface thoroughly under pressure using only the soft brush. (Fig. 8).
3. Then rinse with plenty of lukewarm water (Fig. 9).
4. Finally, allow to air dry or blow dry on a medium setting.

LYMPH FLOW BREAST SHELL

Application instructions for breast shell

It is supplied with a sample size of Amoena Soft Cleanser 30ml and an Amoena Soft Brush. Please clean the breast shell before each use, **especially before the first application.**

1. The breast shell can be cut with scissors according to the breast size. Use the lines provided in the breast shell. To determine the size, place the breast shell on your chest and select the line that corresponds to your breast size. (Fig 10).
2. Place the breast shell directly on the affected breast tissue, with the textured surface facing the skin (Fig 11). The small arrow in the centre of the inside of the shell should point upwards.
3. For optimal effect, the breast shell should be worn under a bra.

Care instructions for breast shell

1. Apply three drops of Amoena Soft Cleanser to the dry, textured surface and then moisten the soft brush with lukewarm water (Fig. 7).
 2. Now clean the textured surface thoroughly using only the soft brush and applying pressure (Fig. 8).
 3. Then rinse with plenty of lukewarm water (Fig. 9).
 4. Finally, allow to air dry or blow dry on a medium setting. The breast shell must not be dried on a heater, in the microwave or in the sun.
- The use of disinfectant should be avoided. Slight discolouration may occur over time but this does not impair the effectiveness of the breast shell.

Storage instructions and shelf life

Textiles: Store in a dry place and protect from sunlight. The maximum shelf life of the compression garment is 36 months and the periode of use is 6 months. On the box/poly bag sticker is the date of manufacture. The total life time of the compression garment is therefore limited to 42 months. If there is a significant change in relevant body dimensions (e.g. due to the success of the therapy, change in weight) of the body part that has already been fitted, a new measurement and a new fitting may be justified even earlier, before the 6-month periode of use has expired.

Pressure pad and breast shell: Store in a dry place and protect from sunlight. The maximum shelf life is 72 months and the periode of use of the pressure pad is 6 months. This depends on the correct handling and use of the products to the usual extent. The total life time of the pressure pad is therefore limited to 78 months. The shelf life is printed with the hourglass symbol on the box label.

Special notes

Page 98.

Disposal instructions

Products should be disposed of in household waste.

Reporting requirement

In the event of any product-related complaints, please contact your specialist medical retailer directly. Only serious incidents that may lead to a significant deterioration of health or death must be reported to the manufacturer and the competent authority of the member state. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation shall apply.

Service information

Still have any questions?

We will be happy to advise you – please give us a call or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2024 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd